


Pessary Selection Chart*

Product Code	Condition and Selection Criteria	Procedentia					Stress Incontinence	Retro Displacement	Incompetent Cervix	Most Common Sizes	Material	Ease of Insertion/Removal	Compatible with Sexual Activity
		Prolapse			Cystocele	Rectocele							
		1 - 2°	3 - 4°	Vaginal Wall									
MXPER	 Ring	X							2½ - 3¾"	Silicone & Metal	Easy	X	
MXPERK	 Ring with knob	X			X	X			2¼ - 3½"	Silicone & Metal	Easy	X	
MXPRS	 Ring with support	X			X				2½ - 3¾"	Silicone & Metal	Easy	X	
MXPRSK	 Ring with support and knob	X			X	X			2¼ - 3½"	Silicone & Metal	Easy	X	
MXPCON	 Incontinence Ring						X		2¼ - 3½"	Silicone & Metal	Easy	X	
MXPSH	 Shaatz	X							2¼ - 3"	Silicone	Medium	X	
MXPDO	 Donut		X		X	X			2½ - 3¾"	Silicone	Medium/Difficult		
MXPREG	 Regula	X		X	X	X			1¼ - 1½"	Silicone & Metal	Easy/Medium		
MXPGE	 Gellhorn Flexible		X						2¼ - 3"	Silicone	Difficult		
MXPEH	Hodge	X					X	X	3½ - 4½" x 2¼ - 2⅝"	Silicone & Metal	Easy/Medium	X	
MXPHS	Hodge with support	X			X	X			2½ - 3⅝"	Silicone & Metal	Easy/Medium	X	
MXPEHK	Hodge with knob	X					X	X	3¼ - 3¾" x 1⅙ - 2¼"	Silicone & Metal	Easy/Medium	X	
MXPHSK	Hodge with knob and support	X			X	X		X	3¼ - 3¾" x 1⅙ - 2¼"	Silicone & Metal	Easy/Medium	X	
MXPRI	Risser	X					X	X	3½ - 4½" x 2¼ - 2⅝"	Silicone & Metal	Easy/Medium	X	
MXPES	Smith	X					X	X	3½ - 4½" x 2¼ - 2⅝"	Silicone & Metal	Easy/Medium	X	
MXPCOND	Incontinence Dish	X					X		2½ - 3⅝"	Silicone	Easy/Medium	X	
MXPCONDS	Incontinence Dish with support	X			X	X			2½ - 3⅝"	Silicone	Easy/Medium	X	
MXPEC	Cube		X	X	X	X			1⅝ - 1¾"	Silicone	Medium	X	
MXPECT	Tandem Cube		X	X	X	X			1⅝ - 1¾"	Silicone	Medium	X	
MXPGS	Gehrung		X		X	X			1⅝ - 1¾" x 2 - 2⅜"	Silicone & Metal	Medium/Difficult	X	
MXPGSK	Gehrung with knob		X		X	X	X		1⅝ - 1¾" x 2 - 2⅜"	Silicone & Metal	Medium/Difficult	X	
MXPINF	Inflatoball		X		X	X			2¼ - 2½"	Latex	Easy	X	

*Source: Viera AJ, Larkins-Pettigrew M. Practical use of the pessary. Am Fam Physician. May 1, 2000;61(9): 2719-26

MILEX® Brand Pessary Product Offerings


P = Prolapse

I = Incontinence

Ring

For mild prolapse.

Product # MXPER (Pessary Only)
Product # MXKPER
(Pessary, Trimo-San, Jel Jector)

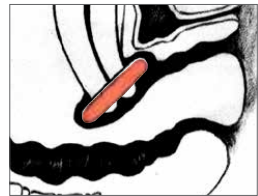


Ring		
Product #	in.	mm
MXPER00	1¾	44
MXPER01	2	51
MXPER02	2¼	57
MXPER03	2½	64
MXPER04	2¾	70
MXPER05	3	76
MXPER06	3¼	83
MXPER07	3½	89
MXPER08	3¾	95
MXPER09	4	102
MXPER10	4¼	108
MXPER11	4½	114
MXPER12	4¾	121
MXPER13	5	127

Ring with Knob

For mild prolapse complicated by stress urinary incontinence.

Product # MXPERK (Pessary Only)
Product # MXKPERK
(Pessary, Trimo-San, Jel Jector)

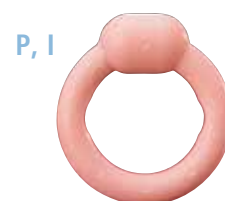

Ring with Knob		
Product #	in.	mm
MXPERK00	1¾	44
MXPERK01	2	51
MXPERK02	2¼	57
MXPERK03	2½	64
MXPERK04	2¾	70
MXPERK05	3	76
MXPERK06	3¼	83
MXPERK07	3½	89
MXPERK08	3¾	95
MXPERK09	4	102
MXPERK10	4¼	108
MXPERK11	4½	114
MXPERK12	4¾	121
MXPERK13	5	127

Ring with Support

For mild prolapse complicated by a mild cystocele.

Product # MXPRS (Pessary Only)
Product # MXKPRS
(Pessary, Trimo-San, Jel Jector)



Ring with Support		
Product #	in.	mm
MXPRS00	1¾	44
MXPRS01	2	51
MXPRS02	2¼	57
MXPRS03	2½	64
MXPRS04	2¾	70
MXPRS05	3	76
MXPRS06	3¼	83
MXPRS07	3½	89
MXPRS08	3¾	95
MXPRS09	4	102
MXPRS10	4¼	108
MXPRS11	4½	114
MXPRS12	4¾	121
MXPRS13	5	127

Incontinence Ring

For stress urinary incontinence.

Product # MXPCON (Pessary Only)
Product # MXKPCON
(Pessary, Trimo-San, Jel Jector)




Incontinence Ring		
Product #	in.	mm
MXPCON00	1¾	44
MXPCON01	2	51
MXPCON02	2¼	57
MXPCON03	2½	64
MXPCON04	2¾	70
MXPCON05	3	76
MXPCON06	3¼	83
MXPCON07	3½	89
MXPCON08	3¾	95
MXPCON09	4	102
MXPCON10	4¼	108
MXPCON11	4½	114
MXPCON12	4¾	121
MXPCON13	5	127

Ring with Support & Knob

For mild prolapse complicated by a mild cystocele and stress urinary incontinence.

Product # MXPRSK (Pessary Only)
Product # MXKPRSK
(Pessary, Trimo-San, Jel Jector)

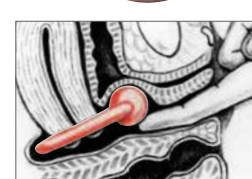


Incontinence Ring w/ Support & Knob		
Product #	in.	mm
MXPRSK00	1¾	44
MXPRSK01	2	51
MXPRSK02	2¼	57
MXPRSK03	2½	64
MXPRSK04	2¾	70
MXPRSK05	3	76
MXPRSK06	3¼	83
MXPRSK07	3½	89
MXPRSK08	3¾	95
MXPRSK09	4	102
MXPRSK10	4¼	108
MXPRSK11	4½	114
MXPRSK12	4¾	121
MXPRSK13	5	127

Shaatz

For mild prolapse complicated by a mild cystocele.

Product # MXPSH (Pessary Only)
Product # MXKPSH
(Pessary, Trimo-San, Jel Jector)


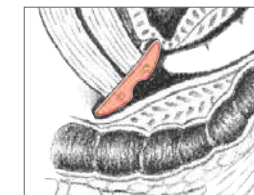


Shaatz		
Product #	in.	mm
MXPSH-1½	1½	38
MXPSH-1¾	1¾	44
MXPSH-2	2	51
MXPSH-2¼	2¼	57
MXPSH-2½	2½	64
MXPSH-2¾	2¾	70
MXPSH-3	3	76
MXPSH-3¼	3¼	83
MXPSH-3½	3½	89
MXPSH-3¾	3¾	95

Hodge

For stress urinary incontinence for a woman who has a very small vaginal introitus.


Product # MXPEH
(Pessary Only)
Product # MXKPEH
(Pessary, Trimo-San, Jel Jector)

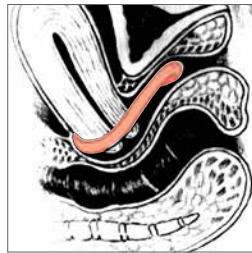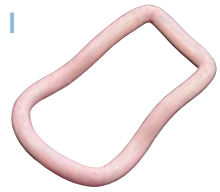

Hodge w/ Support

For stress urinary incontinence with a mild cystocele for a woman who has a very small vaginal introitus.


Product # MXPHS
(Pessary Only)
Product # MXKPHS
(Pessary, Trimo-San, Jel Jector)

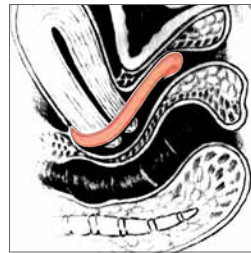

Hodge w/ Knob

For stress urinary incontinence in the woman who has a small vaginal introitus.


Product # MXPEHK
(Pessary Only)
Product # MXKPEHK
(Pessary, Trimo-San, Jel Jector)

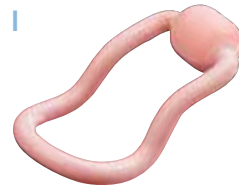

Hodge w/ Support & Knob

For stress urinary incontinence complicated by a mild cystocele in the woman who has a small vaginal introitus.

Product # MXPHSK (Pessary Only)
Product # MXKPHSK
(Pessary, Trimo-San, Jel Jector)

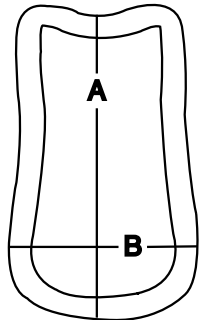

Hodge							
Hodge	w/ Support	w/ Knob	w/ Support & Knob				
Product #	Product #	Product #	Product #	in. (A)	mm (A)	in. (B)	mm (B)
MXPEH00	MXPHS00	MXPEHK00	MXPHSK00	2 $\frac{3}{4}$	70	1 $\frac{3}{4}$	44
MXPEH01	MXPHS01	MXPEHK01	MXPHSK01	3	76	1 $\frac{7}{8}$	48
MXPEH02	MXPHS02	MXPEHK02	MXPHSK02	3 $\frac{1}{4}$	83	1 $\frac{15}{16}$	49
MXPEH03	MXPHS03	MXPEHK03	MXPHSK03	3 $\frac{1}{2}$	89	2	51
MXPEH04	MXPHS04	MXPEHK04	MXPHSK04	3 $\frac{3}{8}$	92	2 $\frac{1}{8}$	54
MXPEH05	MXPHS05	MXPEHK05	MXPHSK05	3 $\frac{3}{4}$	95	2 $\frac{1}{4}$	57
MXPEH06	MXPHS06	MXPEHK06	MXPHSK06	3 $\frac{7}{8}$	98	2 $\frac{3}{8}$	60
MXPEH07	MXPHS07	MXPEHK07	MXPHSK07	4 $\frac{1}{4}$	108	2 $\frac{1}{2}$	64
MXPEH08	MXPHS08	MXPEHK08	MXPHSK08	4 $\frac{5}{8}$	117	2 $\frac{3}{4}$	67
MXPEH09	MXPHS09	MXPEHK09	MXPHSK09	5	127	2 $\frac{3}{4}$	70


Gellhorn

For 3rd degree prolapse/procidentia.

Product # MXPGE (Pessary Only)
Product # MXKPEGE
(Pessary, Trimo-San, Jel Jector)

Also available in:
Short Stem Product # MXPGRS
Rigid Product # MXPGRS
(Pessary Only)

Short Stem Product # MXKPGRS
Rigid Product # MXKPGRS
(Pessary, Trimo-San, Jel Jector)

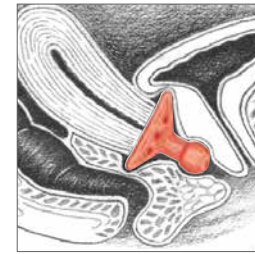


Gellhorn				
Flexible	Rigid	Short Stem		
Product #	Product #	Product #	in.	mm
MXPGE-1 $\frac{1}{2}$	MXPGRS-1 $\frac{1}{2}$	MXPGSS-1 $\frac{1}{2}$	1 $\frac{1}{2}$	38
MXPGE-1 $\frac{3}{4}$	MXPGRS-1 $\frac{3}{4}$	MXPGSS-1 $\frac{3}{4}$	1 $\frac{3}{4}$	44
MXPGE-2	MXPGRS-2	MXPGSS-2	2	51
MXPGE-2 $\frac{1}{4}$	MXPGRS-2 $\frac{1}{4}$	MXPGSS-2 $\frac{1}{4}$	2 $\frac{1}{4}$	57
MXPGE-2 $\frac{1}{2}$	MXPGRS-2 $\frac{1}{2}$	MXPGSS-2 $\frac{1}{2}$	2 $\frac{1}{2}$	64
MXPGE-2 $\frac{3}{4}$	MXPGRS-2 $\frac{3}{4}$	MXPGSS-2 $\frac{3}{4}$	2 $\frac{3}{4}$	70
MXPGE-3	MXPGRS-3	MXPGSS-3	3	76
MXPGE-3 $\frac{1}{4}$	MXPGRS-3 $\frac{1}{4}$	MXPGSS-3 $\frac{1}{4}$	3 $\frac{1}{4}$	83
MXPGE-3 $\frac{1}{2}$	MXPGRS-3 $\frac{1}{2}$	MXPGSS-3 $\frac{1}{2}$	3 $\frac{1}{2}$	89
MXPGE-3 $\frac{3}{4}$	MXPGRS-3 $\frac{3}{4}$	MXPGSS-3 $\frac{3}{4}$	3 $\frac{3}{4}$	95




Regula

For uterine prolapse (1st and 2nd degree).

Product # MXPREG (Pessary Only)
Product # MXKPREG
(Pessary, Trimo-San, Jel Jector)

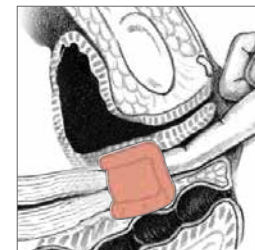

Regula Folding						
Product #	in. (A)	mm (A)	in. (B)	mm (B)	in. (C)	mm (C)
MXPREG00	1	25	1 $\frac{15}{16}$	49	1 $\frac{3}{8}$	35
MXPREG01	1 $\frac{1}{16}$	27	2	51	1 $\frac{1}{2}$	38
MXPREG02	1 $\frac{1}{8}$	29	2 $\frac{1}{8}$	54	1 $\frac{7}{8}$	41
MXPREG03	1 $\frac{1}{4}$	32	2 $\frac{3}{16}$	56	1 $\frac{3}{4}$	44
MXPREG04	1 $\frac{3}{8}$	35	2 $\frac{1}{4}$	57	1 $\frac{7}{8}$	48
MXPREG05	1 $\frac{7}{16}$	37	2 $\frac{3}{8}$	60	2	51
MXPREG06	1 $\frac{1}{2}$	38	2 $\frac{5}{8}$	67	2 $\frac{1}{8}$	54
MXPREG07	1 $\frac{5}{8}$	41	2 $\frac{7}{8}$	73	2 $\frac{1}{4}$	57
MXPREG08	1 $\frac{3}{4}$	44	3 $\frac{1}{8}$	79	2 $\frac{3}{8}$	60

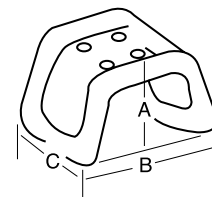


Cube

Support for 3rd degree prolapse/procidencia.
Should be removed nightly.

Product # MXPEC (Pessary Only)
Product # MXKPEC
(Pessary, Trimo-San, Jel Jector)


Also available in:
Cube with Drainage Holes
Product # MXPECH (Pessary Only)
Product # MXKPECH
(Pessary, Trimo-San, Jel Jector)

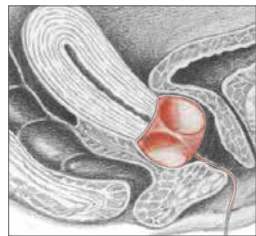

Tandem-Cube


Support for 3rd degree prolapse/procidencia when single cube provides inadequate support.
Should be removed nightly.

Product # MXPECT (Pessary Only)
Product # MXKPECT
(Pessary, Trimo-San, Jel Jector)

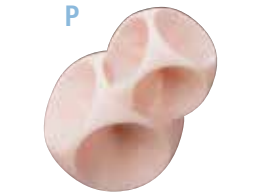

Cube			
Cube	w/ Holes		
Product #	Product #	in.	mm
MXPEC00	MXPECH00	1	25
MXPEC01	MXPECH01	1 ¹ / ₁₆	30
MXPEC02	MXPECH02	1 ¹ / ₈	35
MXPEC03	MXPECH03	1 ¹ / ₂	38
MXPEC04	MXPECH04	1 ¹ / ₄	41
MXPEC05	MXPECH05	1 ³ / ₄	44
MXPEC06	MXPECH06	2	51
MXPEC07	MXPECH07	2 ¹ / ₄	57


Tandem Cube		
Product #	in.	mm
MXPECT2-0	1 ¹ / ₈ & 1	35 & 25
MXPECT3-1	1 ¹ / ₂ & 1 ¹ / ₁₆	38 & 30
MXPECT4-2	1 ¹ / ₈ & 1 ¹ / ₈	41 & 35
MXPECT5-3	1 ³ / ₄ & 1 ¹ / ₂	44 & 38
MXPECT6-4	2 & 1 ¹ / ₈	51 & 41
MXPECT7-5	2 ¹ / ₄ & 1 ¹ / ₄	57 & 44

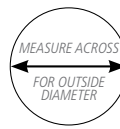


Risser

Designed for patient with shallow pubic notch. Has larger weight bearing zone to support vaginal aspects of pubis. Helpful in cases of repeated miscarriages due to an incompetent cervix by repositioning the weight of the growing fetus.

Product # MXPRI (Pessary Only)
Product # MXKPRI
(Pessary, Trimo-San, Jel Jector)

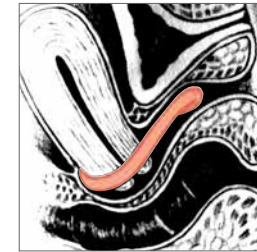


Risser				
Product #	in. (A)	mm (A)	in. (B)	mm (B)
MXPRI00	3 ¹ / ₈	79	2	51
MXPRI01	3 ¹ / ₄	83	2 ¹ / ₈	54
MXPRI02	3 ¹ / ₂	89	2 ¹ / ₄	57
MXPRI03	3 ³ / ₄	95	2 ³ / ₈	60
MXPRI04	4 ¹ / ₄	108	2 ¹ / ₂	64
MXPRI05	4 ¹ / ₂	114	2 ³ / ₄	67
MXPRI06	4 ³ / ₄	121	2 ⁷ / ₈	70
MXPRI07	5	127	2 ⁷ / ₈	73
MXPRI08	5 ¹ / ₂	140	3	76
MXPRI09	5 ³ / ₄	146	3 ¹ / ₈	79

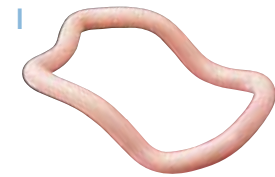

Incontinence Dish

For stress urinary incontinence with mild prolapse.


Product # MXPCOND (Pessary Only)
Product # MXKPCOND
(Pessary, Trimo-San, Jel Jector)

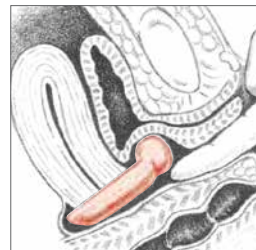

Incontinence Dish with Support

For stress urinary incontinence with mild prolapse and a mild cystocele.

Product # MXPCONDS (Pessary Only)
Product # MXKPCONDS
(Pessary, Trimo-San, Jel Jector)

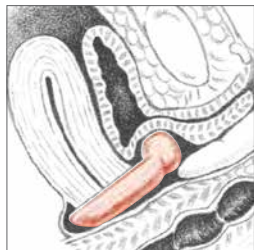


Incontinence Dish			
Dish	Dish w/ Support		
Product #	Product #	in.	mm
MXPCOND00	MXPCONDS00	2 ¹ / ₈	55
MXPCOND01	MXPCONDS01	2 ¹ / ₄	60
MXPCOND02	MXPCONDS02	2 ¹ / ₂	65
MXPCOND03	MXPCONDS03	2 ³ / ₄	70
MXPCOND04	MXPCONDS04	3	75
MXPCOND05	MXPCONDS05	3 ¹ / ₈	80
MXPCOND06	MXPCONDS06	3 ¹ / ₄	85


Smith

Modification of Hodge pessary. For stress urinary incontinence designed for patients with a well-defined pubic-notch. Helpful in cases of repeated miscarriages due to incompetent cervix by repositioning the weight of the growing fetus.

Product # MXPES (Pessary Only)
Product # MXKPES
(Pessary, Trimo-San, Jel Jector)

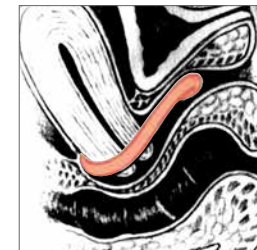



Smith				
Product #	in. (A)	mm (A)	in. (B)	mm (B)
MXPES00	3 ¹ / ₈	79	2	51
MXPES01	3 ¹ / ₄	83	2 ¹ / ₈	54
MXPES02	3 ¹ / ₂	89	2 ¹ / ₄	57
MXPES03	3 ³ / ₄	95	2 ³ / ₈	60
MXPES04	4 ¹ / ₄	108	2 ¹ / ₂	64
MXPES05	4 ¹ / ₂	114	2 ³ / ₄	67
MXPES06	4 ³ / ₄	121	2 ⁷ / ₈	70
MXPES07	5	127	2 ⁷ / ₈	73
MXPES08	5 ¹ / ₂	140	3	76
MXPES09	5 ³ / ₄	146	3 ¹ / ₈	79

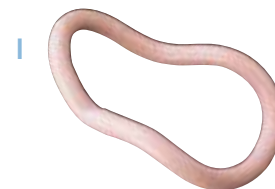

Gehung with Knob/Folding

For cystocele with or without mild rectocele complicated by stress urinary incontinence.

Product # MXP GSK (Pessary Only)
Product # MXKPGSK
(Pessary, Trimo-San, Jel Jector)

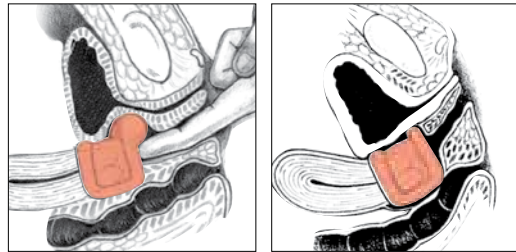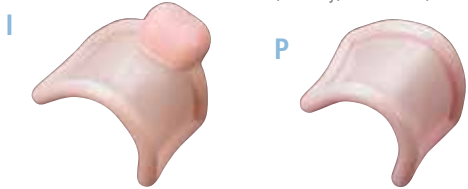


Gehung/Folding

Support for cystocele and rectocele. It is also more adaptable in cases of procidentia where the uterus tends to herniate when other pessaries are used.

Product # MXP GS (Pessary Only)
Product # MXKPGS
(Pessary, Trimo-San, Jel Jector)

Gehung							
Gehung w/ Knob/Folding	Gehung Folding						
Product #	Product #	in. (A)	mm (A)	in. (B)	mm (B)	in. (C)	mm (C)
MXP GSK00	MXP GS00	1 1/16	27	1 1/2	38	1 3/8	35
MXP GSK01	MXP GS01	1 1/8	29	1 3/4	44	1 1/2	38
MXP GSK02	MXP GS02	1 3/8	35	2	51	1 5/8	41
MXP GSK03	MXP GS03	1 1/2	38	2 1/8	54	1 3/4	44
MXP GSK04	MXP GS04	1 5/8	41	2 1/4	57	1 7/8	48
MXP GSK05	MXP GS05	1 3/4	44	2 3/8	60	2	51
MXP GSK06	MXP GS06	1 7/8	48	2 1/2	64	2 1/8	54
MXP GSK07	MXP GS07	2	51	2 5/8	73	2 1/4	57
MXP GSK08	MXP GS08	2 1/8	54	3	76	2 3/8	60
MXP GSK09	MXP GS09	2 1/4	57	3 1/8	79	2 3/4	64

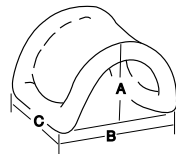

Trimo-San Vaginal Jelly*


Trimo-San is a recommended adjunct for pessary wearers. For complete information, see product enclosure.

*Not sold outside U.S.

Has not been tested for use during pregnancy.



Trimo-San Vaginal Jelly		
Product #	Description	Size
MX5030	Trimo-San Jelly with Jector	4 oz
MXAPP	Jector Replacement	BX


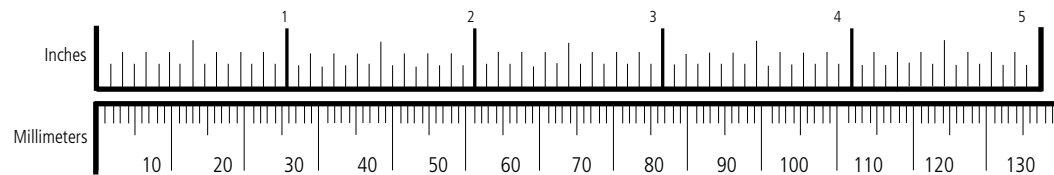


Note: Shaded areas show the minimum assortment necessary to fit approximately 85% of patients.

Donut

Support for 3rd degree prolapse/procidentia. Not to be deflated during insertion or removal. See Inflatoball if a deflatable pessary is necessary.



Product # MXP DO (Pessary Only)
Product # MXKPDO
(Pessary, Trimo-San, Jel Jector)

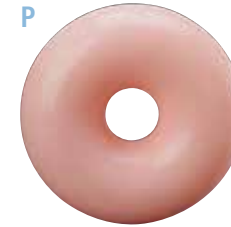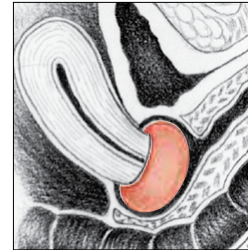

Inflatoball™

Inflatable for 3rd degree prolapse/procidentia with or without a mild cystocele and/or rectocele.

Product # MXP INF (Pessary Only)
Product # MXKINF
(Pessary, Trimo-San, Jel Jector)

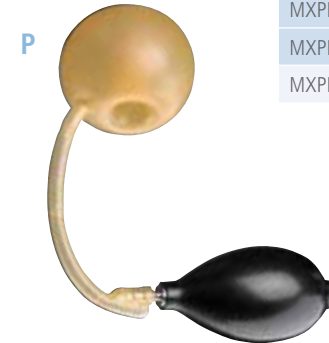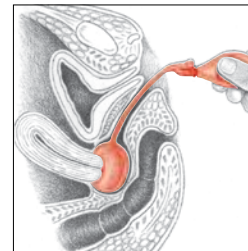

Donut		
Product #	in.	mm
MXP DO00	2	51
MXP DO01	2 1/4	57
MXP DO02	2 1/2	64
MXP DO03	2 3/4	70
MXP DO04	3	76
MXP DO05	3 1/4	83
MXP DO06	3 1/2	89
MXP DO07	3 3/4	95

Inflatoball—Size		
Product #	in.	mm
MXP INF S	2	51
MXP INF M	2 1/4	57
MXP INF L	2 1/2	64
MXP INF XL	2 3/4	70



CooperSurgical is the leading company dedicated to providing medical devices and procedure solutions that improve health care delivery to women regardless of clinical setting. Our company is fostering that position through expansion of its core businesses and introduction of advanced technology-based products which aid clinicians in the management and treatment of commonly seen conditions.

203.601.5200 | 800.243.2974 | coopersurgical.com | 95 Corporate Dr. | Trumbull, CT 06611